

FloorBed 2 Head and Foot Bumper Instruction Manual

(Bumpers include handset pocket)

IFU-FL1-005EN Rev 01

Contents

Title	Page
Welcome	2
General warnings	3
1. Safety Instructions	3
2. Intended use	3
3. Technical specification	4
4. Positioning	4
5. Fitting	4
6. Fitting for use by child patients	6
7. Siderail fitting	6
8. Functionality check	6
9. Cleaning and disinfection	7
10. Storage	7
11. Guarantee	7
12. Disposal	7
13. Table of symbols	7
14. Contact details	7

Welcome

Dear Customer,

Thank you for purchasing an Accora healthcare product. Before using this accessory, you must read and understand all the instructions in this manual. All actions and handling of the accessory must be performed in accordance with the instructions in this manual.

Please ensure that the manual is available to users and operators throughout the accessory's service life.

If you need further information, please contact us. See section 14 for region specific contact details.

This instruction manual applies to the FloorBed 2 head and foot end bumpers.

Before using this accessory, you must become acquainted in detail with this manual. All actions and handling of the FloorBed head and foot end bumpers must be performed in accordance with the instructions in this manual.

Any actions that are inconsistent with the manual are performed at your own risk and Accora shall not be liable for any injury or damage. Please ensure that the manual is available to operators throughout the use of the FloorBed head and foot end bumpers.

FloorBed head and foot end bumpers should be regularly checked by the carer/user to ensure the setup is correct and it is working as intended.

Design Policy and Copyright

® And ™ are trademarks belonging to Accora Ltd unless otherwise stated. As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice. © Accora Ltd 2021.

GENERAL WARNINGS

1. Keep this Instruction Manual available for future reference.
2. These instructions must be observed to ensure the safe and effective use of this FloorBed head and foot end bumper and the safety of users and carers and those around.
3. The FloorBed head and foot end bumper must be fitted, positioned and used in accordance with these instructions.
4. The safety features for using the FloorBed head and foot end bumper and instructions for use must be strictly observed.
5. The FloorBed head and foot end bumpers must not be exposed to smoke, naked flame, extreme temperature, flammable gases or other hazardous substances or situations.
6. Accora shall not be held liable for any damage, injuries or accidents arising from unauthorised modifications, non-genuine spare parts, negligence or use that is at variance with this manual.
7. Do not move the bed by pulling or pushing on the FloorBed head and foot end bumpers.
8. Do not modify these FloorBed head and foot end bumpers.
9. Patients, or users, should be risk assessed to ensure they can understand this manual and use the FloorBed head and foot end bumper safely without risk to themselves or others.

1. Safety Instructions

1. Before using the FloorBed head and foot end bumper, you must read the manual and use this device in accordance with it.
2. The user should understand the functionality of the FloorBed head and foot end bumpers before fitting them onto the bed.
3. Do Not Use the FloorBed head and foot end bumpers if damaged or faulty as this could lead to injury.
4. A risk assessment must be carried out before the bed is used by a patient.
5. When replacing missing parts, only original materials and components must be used.

2. Intended use

Subject to a risk assessment, the FloorBed head and foot end bumpers may be fitted onto the beds defined below to help maintain, improve, compensate or alleviate the condition of the patient. They will provide protection and reduce the risk of injury.

Description	User Group
FloorBed 2 NSB-0-FL2-200	Adult patients Min 146cm height
FloorBed 2 Junior comprises FloorBed 2 NSB-0-FL2-200 + Head and Footboard Bumpers BMHNE-0-FL4-500 + Safety Sleeve SLE-0-FL4-100 + Protection Plates PROPL-0-FL4-500	Child patients Min 75cm height Max 155cm height

When the Bumper Set is fitted with the Safety Sleeve and Protection Plates, in accordance with their Instructions for Use, the bed is suitable for children as shown in the table above.

It is essential to consult Accora in advance if you wish to use the bed for any purpose outside the use detailed in this manual.

3. Technical specification

Description	Value
Part number	BMHNE-0-FL4-500
Overall dimensions	900 x 800 x 100mm 37 x 32 x 4in
Fits bed	FloorBed 2
Compatibility	NSB-0-FL2-200
Weight	3.5kg/10lbs

4. Positioning

WARNING

The FloorBed head and foot end bumper must be fitted with due consideration to the best practice described below. Failure to do so may result in entrapment and serious injury.

The head and foot end bumpers should be positioned such that the inside faces of the head and foot ends of the bed are covered by the fully padded side of the bumper as shown below.



5. Fitting

WARNING

Fitting **MUST** be carried out by a competent person.

All functions **MUST** be tested and approved by a competent person after fitting.


Fitting **MUST** take place in a clear, uncluttered area and children and pets should be kept away.

The FloorBed head and foot end bumpers must not be fitted with a patient in the bed.

1. Check the delivery is complete and undamaged. Do not use if any damage has occurred. You should have received:
 - a. Head end bumper.
 - b. Foot end bumper.
 - c. 4 x M6 x 35mm joint connector bolt and nut as shown below.




2. Remove the mattress from the bed if fitted.
3. Remove the pins and R clips which locate the mattress platform on the head and foot end of the bed as shown below.


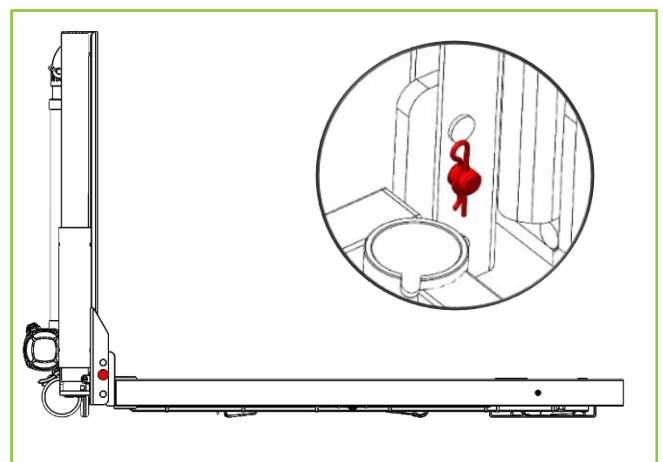


4. Fit the new joint connector bolts and nuts in place of the pins and R clips. Tighten securely.


5. Pull the bumper down over the headboard ensuring that the inside of the headboard is covered by the padded side of the bumper and that the zipped compartment is on the outside of the head end of the bed.

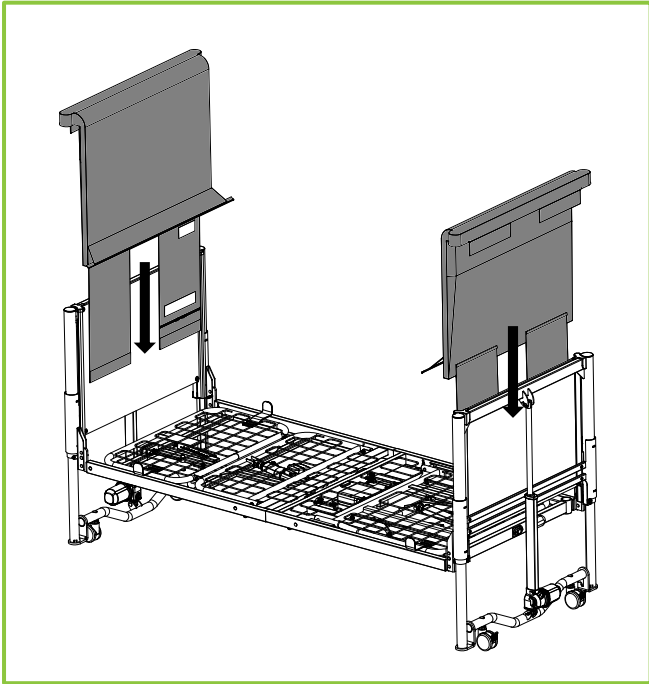


6. Underneath the bumper are two metal tabs (1). When fitting the bumper, these must locate into the slot on the headboard and footboard (2).

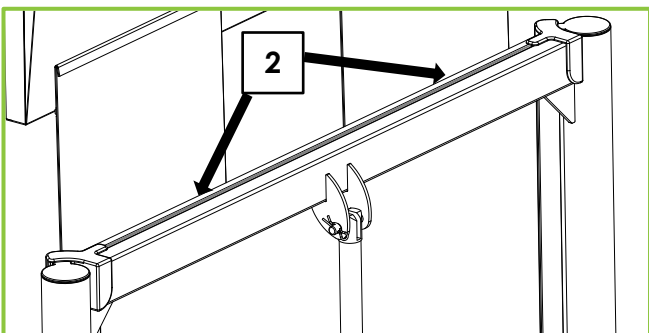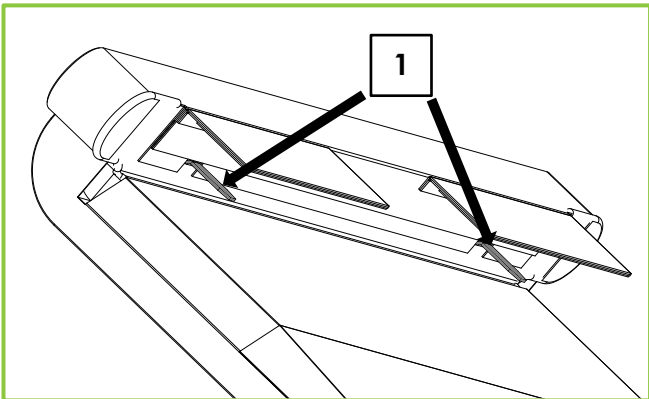

7. Ensure that the tabs are located correctly, and the bumper is sitting squarely on the top of the headboard. Fold up each flap (3) and secure the Velcro under the small flap at the top of the headboard (4).

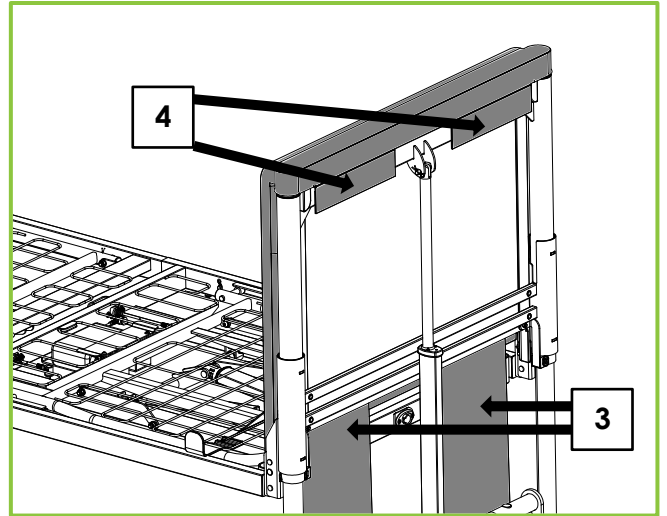


8. Repeat steps 5-7 for the foot end of the bed.

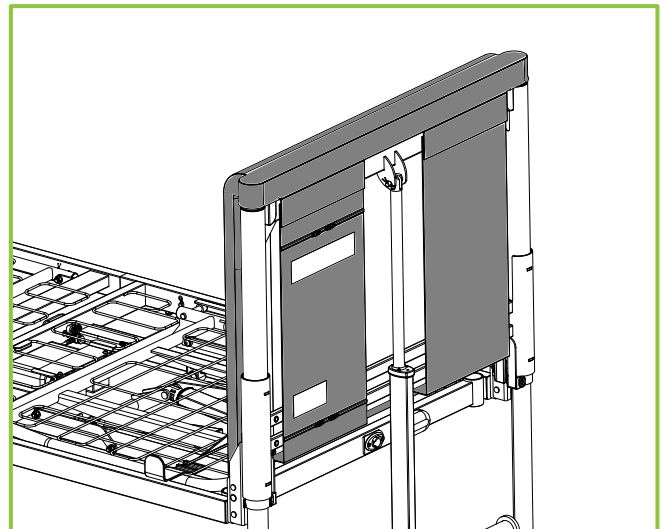

9. If the bed to be used with a safety sleeve, note that the safety sleeve is not zipped to the bumper.


10. Bed with bumpers fitted.

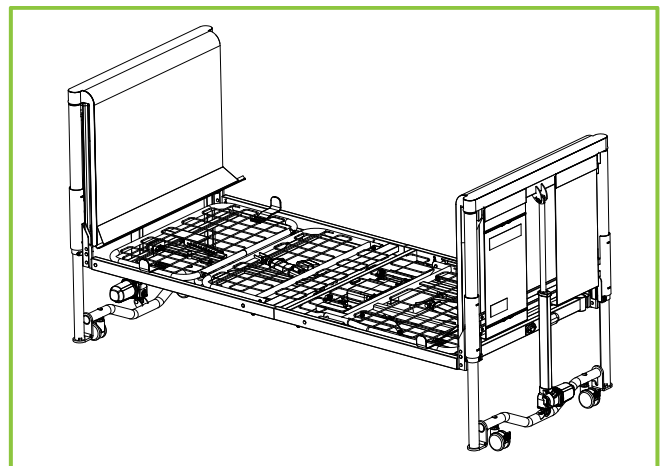

6. Fitting for use by child patients

If the bed is being configured for use with a child patient the following steps must be made in addition to those detailed above to make the bed appropriate for use with children.

1. Check that the safety sleeve and head and foot end bumpers are installed correctly according to their respective manuals.
2. Locate the handset lead where it comes out of the Safety Sleeve at the head end of the bed.
3. The bumper has a pocket with an opening which is zipped at top and bottom. There is a horizontal strip of Velcro inside the pocket.
4. Feed the handset through the bottom hole of the pocket. Part the internal Velcro and feed the handset through the pocket and out of the top pocket as shown.
5. Pull the handset through the pocket as far as it will go. Close the bottom zip up to the handset lead and press the internal Velcro together against the lead.

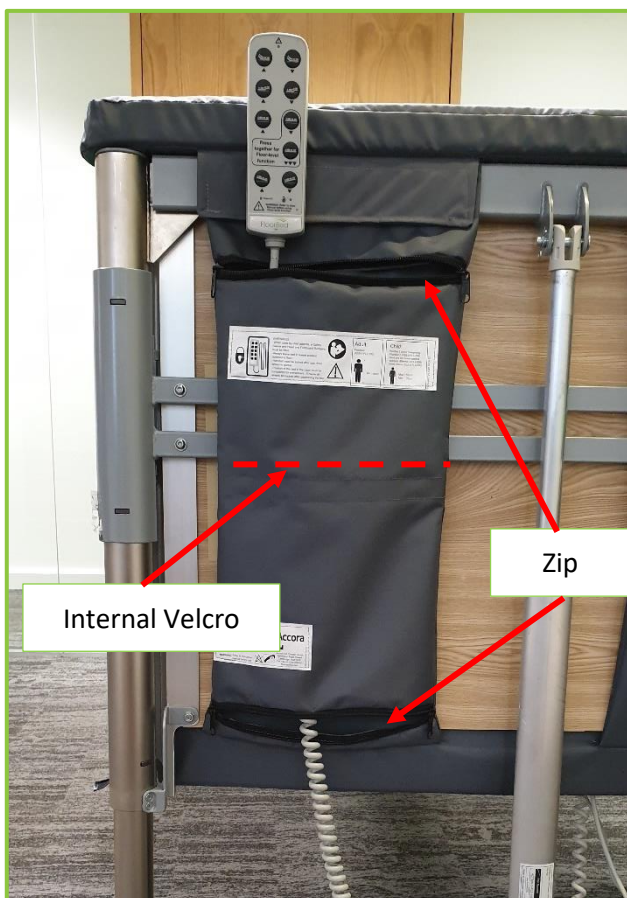

6. If it is necessary to securely lock away the handset when it is not being used, the handset should be locked (see the relevant bed manual for details of this procedure) and placed in the top section of the pocket. The zip sliders should then be pulled together, and the provided padlock can be used to lock the two sliders together. (When unlocked the padlock should be kept in a secure place out of children's reach.)

7. Siderail fitting (If required)

1. If siderails are to be used with the bed they must be fitted after the safety sleeve and bumper have been fitted.
2. Refer to the siderail instruction manual for full fitting instructions.

8. Functionality Check

WARNING

Fitting **MUST** be carried out by a competent person.

All bed functions **MUST** be tested and approved by a competent person after assembly.

Functionality check must not be carried out with the bed occupied.

Carry out following functions of the bed and check for unexpected noises or movements:

1. Raise and lower the bed
2. Raise and lower the backrest
3. Raise and lower the legrest
4. Raise and lower the siderail (if fitted)

9. Cleaning and Disinfection

WARNING

Cleaning and disinfection **MUST** be carried out by a competent person.

All functions **MUST** be tested and approved by a competent person after cleaning or disinfection.

The FloorBed head and foot end bumpers **MUST** be cleaned and disinfected before re-using the bed for a different patient.

To disinfect the FloorBed head and foot end bumpers, only use detergents designed for use in healthcare. Do not use abrasives, scourers or other materials that could damage the material. Do not use corrosives, caustics or strong acids.

Do not use products that could alter the structure or behaviour of the plastics (petrol etc.).

10. Storage

For problem-free storage we recommend:

1. FloorBed head and foot end bumpers should be stored in a temperature between -20°C to +50°C, -4°F to +122°F
2. FloorBed head and foot end bumpers should be stored in a relative humidity (non-condensing) between 30% and 75%














11. Guarantee

Model number	BMHNE-0-FL4-500
Warranty period	2 years

12. Disposal

In the event of the disposal of the FloorBed head and foot end bumpers, end-of-life parts must be disposed of in accordance with current environmental regulations.

13. Table of Symbols

	Warning, beware of potential hazard – refer to instructions for use
	Refer to instructions for use
	Model number
	Lot number
	Manufactured date
	Manufacturer
	Do not iron
	Do not tumble dry
	Do not bleach
	Wipe down
	Lockable handset storage location
	Patient Length (Adult)
	Patient Length (Child)

14. Contact Details

Address	Accora Ltd. Charter House Barrington Road, Orwell Cambridge, SG8 5QP, UK
Telephone	+44 (0) 1223 206100
Email	info@accora.care
Website	www.accora.care

Accora



Accora
Barrington Road
Orwell
Cambridge
SG8 5QP
United Kingdom

T: +44 (0)1223 206100
info@accora.care
www.accora.care