

Folding Safety Mat

VERSION ONE | NOVEMBER 2015

IFU-FL1-010EN Rev 01

Specification

The Folding Safety Mat is a mat that provides users with a cushioned platform that is used next to or against the bed so as to protect users from falling and injuring themselves on the hard floor in the event of a bed fall.

Part Number: SAFTF-0-AL1-000


Part Description: Safety mat, folding

Installation Guide

1. Unpack the Safety Mat from the plastic bag.
2. Remove the Velcro strap and keep it safely stored. This strap will be very useful for transporting the mat at a later date.



3. Extend the mat to full length by unfolding it, and place the safety mat next to the bed.

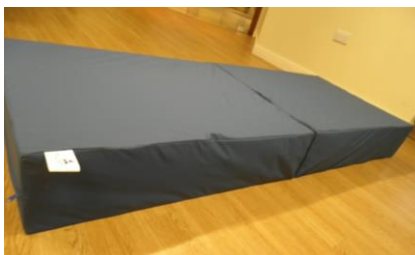

4. When the mat is not in use then fold up and make sure it is not in the way.

Regulations

The Folding Safety Mat are a Medical Device Class



The Folding Safety Mat is tested to BS 7175 Crib 5.

Cleaning Recommendations

The Safety Mat can be wiped down with a mild detergent. Place some of the detergent on a damp cloth and wipe the mat down.

If the foam is removed then the cover can be placed in a clothes washing machine. Please see washing instructions:



Storage & Care

Store the product by folding it into three parts. Place the Velcro strap round it so that it does not unfold.

Place in the original bag so that it is protected from any humidity/dirt.

Do not allow sharp objects to penetrate the cover coating (including hypodermic needles).

Do not store in damp conditions.

Guarantee

The warranty period for the Folding Safety Mat is 12 months. We require a purchase order for any parts despatched, warranty or otherwise. Any issues regarding warranty should be addressed in the first instance to your supplier.